figure 1

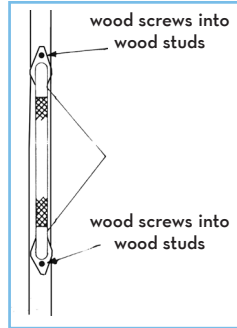
figure 2

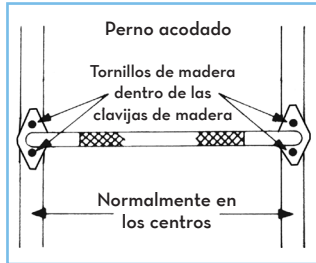
figura 1

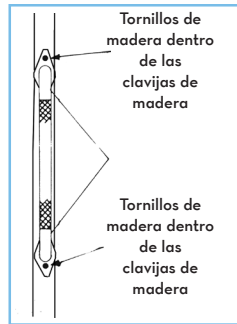
figura 2

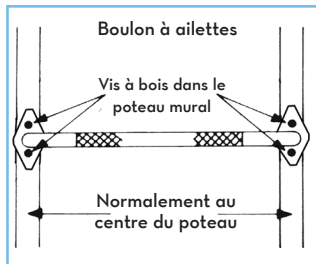
figure 1

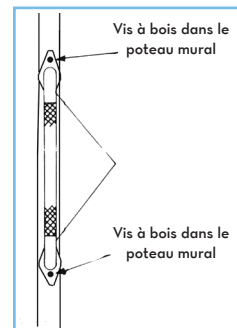
figure 2

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive CE
MEDICAL DESIGN & MANUFACTURING

© 2005 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

grab bar

barra de sostén

barre d'appui

drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

mounting instructions

NOTE: Mounting flanges of grab bar must be aligned to mount to wall studs. Wood screws must go through the wall board or tile and attach to wood wall studs for solid support. For plastic tub and shower enclosures, drill through wall and mount grab bar to wall studs.

1. Place grab bar on wall in line with wall studs and mark location of screw holes. (Horizontal Attachment See Figure 1, Vertical Attachment See Figure 2.) After deciding on type of attachment, drill small hole to confirm the location of the stud.

CAUTION: Wood screws should only be used in wooded wall studs.

2. Determine for each flange whether to use wood screws or toggle bolts.

WARNING: At least two wood screws in each flange must be attached to wall studs or grab bar may pull out and serious injury could result

3. Use masonry bit to drill through tile or grout. For wood screws, drill small pilot hole into wood studs.
4. After holes are drilled, place grab bar on wall and attach with wood screw. If using toggle bolts, make sure bolt is attached to grab bar flange before being forced into hole.

NOTE: Use caulking around grab bar flanges to prevent moisture from seeping in.

NOTE: Clean with mild soap and rinse with warm water.

5. Tighten all screws and bolts with screwdriver. Check periodically for tightness. Never use grab bar if any looseness is detected as it could result in personal injury.

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