

NEW

PRODIGY®
AutoCode

**No Coding
English & Spanish
Talking**

Blood Glucose Monitoring System



Actual Size Shown

***Easy to Use, Accurate, Audible Results
in just 7 Seconds!***



Diagnostic Devices, Inc.

www.prodigymeter.com



No Coding, English & Spanish Talking, Blood Glucose Monitoring System. Great for the Low Vision. Accurate. Easy. Fast!

Test Strip Port: NEW easy to locate strip port! The meter automatically turns ON when the user inserts the test strip. New improved test strips are more accurate and works on ALL AutoCode meters.

**NEW
Meter
Design**

Time and Date.

Test Results Readout: Test results are displayed in large easy to read size.

Speaker: A clear and pleasant voice talks to you.

Actual Size.

New AutoCode meter shape fits comfortably in your hand. Now in black & silver.

Voice On/Off: Indicates "talking" feature is on/off. The Prodigy AutoCode® speaks the test results in either English or Spanish!

Main Button: Enter memory recall mode. Easy one button operation.

Mini USB Port: Uses any standard USB Cable. NO special cables needed! Download the meters test results to your computer with Prodigy®'s FREE software.

The **Prodigy AutoCode®** Meter requires **NO-Coding**, which means fewer steps and avoids the danger of false results caused by coding errors. Simply insert the test strip into the strip port, located at the top of the meter, and the Prodigy AutoCode® automatically turns on and will "talk" the user through the testing steps. Making it easy to use, while providing safe, accurate test results every time.

Audible. An essential tool for the visually impaired. The "talking" feature will let the user know that the meter is on and then direct the user to apply a drop of blood to the test strip. Once the absorbent channel on the test strip is full, the user will **hear** their test results in just 7 seconds, with time and date in either English or Spanish.


Memory and Data Management. Prodigy AutoCode® can store up to 120 tests results with date and time, and gives 7, 14, 28 day averages. Uses standard USB cable. Allowing unlimited memory with Prodigy®'s FREE Data Management Software. Great for your Diabetes healthcare professional to track changes in your blood glucose level over time.

Greater Comfort. Prodigy AutoCode® offers you the option to test with blood from your palms, forearms, upper-arms, thighs and calves, which have fewer nerve endings than your fingertips.

Smaller Blood Sample Required. Prodigy AutoCode® requires tiny drop of blood - smaller than most meters available today. This allows you to use extra-fine gauge lancets and significantly reduce your pain and discomfort.

Teamwork. With no coding and audible features, this makes the Prodigy AutoCode®, **the easiest and safest blood glucose monitoring system to use.** Another great advantage is that promotes "team work" ... the ability to "hear" your child's, or your spouse's blood sugar reading from across the room without having to ask. With its compact size, the user can slip the AutoCode meter into their pocket or purse easily and take it with them anywhere.

DDI is dedicated to making safe, accurate, innovative and affordably priced meters and test strips! It's our passion to help everyone "Live Well with Diabetes™."

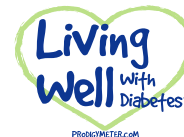

To learn more about the **Prodigy AutoCode®** call us toll free **1-800-366-5901** or go on-line to www.prodigymeter.com



Diagnostic Devices, Inc.
9300 Harris Corners Parkway,
Suite 450
Charlotte, NC 28269