



SIZING:

Measure around waist just below the navel and measure hips at the widest point.

Size	Fits Waist	Fits Hips
XSmall	26 - 29"	30 - 33"
Small	30 - 33"	34 - 37"
Medium	34 - 37"	38 - 41"
Large	38 - 41"	42 - 45"
XLarge	42 - 45"	46 - 49"
XXLarge	46 - 49"	50 - 53"
XXXLarge	50 - 53"	54 - 57"



P.O. Box 277810
Miramar, FL 33027-7810
(800) 327-4110 Tel
(800) 874-4117 Fax
www.flaorthopedics.com

Safe-T-Lift® DX Occupational Back Support

Model No: 70-120
Sizes: XSmall - XXXLarge
Color: Black

Ideal for industrial use, material handling, warehouse and shipping occupations, movers, etc.

Product Features:

- Polypropylene stays provide extra support
- Wide support base made from breathable spandex
- Safe-T-Fit™ System provides a scaled, conforming fit; support base height is scaled to body size

How to Apply:

Bring the slip locks on the shoulder straps all the way down and then insert arms through shoulder straps and loosen side pull straps. Place left flap just below navel, overlay left flap with right flap and press down to close hook and loop fastener. Adjust shoulder straps until they are snug but not tight. When ready to lift, grab the two side pull straps and place them together in the center of the support. By moving the adjustable side pull straps, you can increase or decrease the amount of tension needed. When finished lifting, move the side pulls back to the passive position.

Care Instructions:

For best results, hand wash and air dry support. If you machine wash, fasten all closures on the support and wash in cold water on gentle cycle with mild soap. DO NOT place in dryer.