

Sizing Guide

Measure the hips or waist (select the larger of the two) as follows:

- Step 1:** Measure hip bone to hip bone and over the abdomen
- Step 2:** Double the largest measurement obtained in Step 1 and add two inches
- Step 3:** Using this figure, make product size selection from the chart below



EXAMPLE

Hip bone to hip bone = 19 inches
 19 inches x 2 inches =
 38 inches + 2 inches = 40 inches

40 inches = Medium/Large brief

PRODUCT SIZING GUIDE

BRIEF SIZE SELECTION

MEDLINE BRAND	MOLICARE® CLASSIC PLUS
SMALL = 20" – 31" (green or light blue)	SMALL = 20" – 34" (light blue)
MEDIUM = 32" – 44" (white)	MEDIUM/LARGE = 27" – 47" (white)
REGULAR = 34" – 56" (purple)	LARGE/X-LARGE = 39" – 59" (dark blue)
LARGE = 45" – 58" (blue)	MOLICARE® AIR ACTIVE
X-LARGE = 59" – 66" (beige)	SMALL = 24" – 35" (white)
XX-LARGE = 60" – 69" (green)	MEDIUM = 35" – 47" (white)
BARIATRIC = 65" – 90" (beige)	LARGE = 47" – 59" (white)
	X-LARGE = 59" – 69" (white)

1-800-MEDLINE (633-5463)

www.medline.com

Medline, Protection Plus, Ultrasorbs, Asorbs, Aloetouch and DryTime are registered trademarks of Medline Industries, Inc. Restore, Comfort-Aire, Baribrief, Soft-Spun and Remedy are trademarks of Medline Industries, Inc. MoliCare, MoliForm and MoliMed are registered trademarks of PAUL HARTMANN AG. SuperCore is a registered trademark of McAirIaid's Vliesstoffe GmbH & Co.

Medline Canada 1-800-396-6996

www.medline.ca | canada@medline.com

Not all products are currently available for sale in Canada.

Please contact your Account Representative for additional information.

We reserve the right to correct any errors that may occur within this brochure.