MICHAEL GRAVES
active living
COLLECTION™

Bath & Shower Seat With Back Instructions & Warranty

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EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA
Made in China

Item # MG12170

Before assembly, insure that all parts and quantities match parts list. Inspect all components for shipping damage. If any parts are determined to be missing or damaged, contact your dealer/retailer for replacements.

Warning!

DO NOT use this equipment without first reading and understanding its instruction sheet. If you are unable to understand these instructions, contact a healthcare professional, dealer, or technical person before installing. Otherwise, injury or damage may occur. These shower seats are not designed to support the total weight of an individual. Use arms for assistance only.

Assembly Instructions:

1. Place seat upside down on a flat surface. Insert leg into the corner recess of the seat by applying a firm downward pressure until leg is fully engaged. Repeat step #1 for all 4 legs.
2. For models with a back place back in the slot, which is in the rear of the seat, and insert back until engaged.

Adjusting Leg Extensions:

1. Select 1 of the 5 adjustment holes located on the lower leg frame.
2. Depress the snap button and slide the adjustable leg up or down to the desired height.
3. Ensure that the snap buttons fully protrude through the same respective adjustment hole of each leg extension.
4. Repeat STEPS 2-3 for the other legs.
5. Make sure all four legs are adjusted to the same height so that the bath bench sits level.

Care And Maintenance:

1. Do not use an abrasive cleaner. Use a non-abrasive cleaner with warm water.
2. Periodically inspect all components for damage or looseness.

Limited Lifetime Warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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