**RESPIRONICS®**

**MicroElite™**

**Compressor Nebulizer System**

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**CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician or licensed healthcare professional.

Contents: (1) MicroElite Compressor with Instructions  
(1) Nebulizer with Tubing

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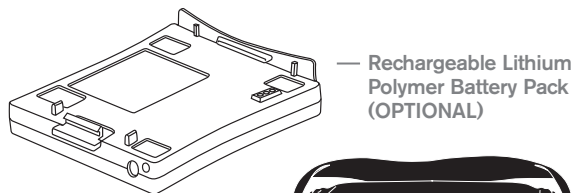
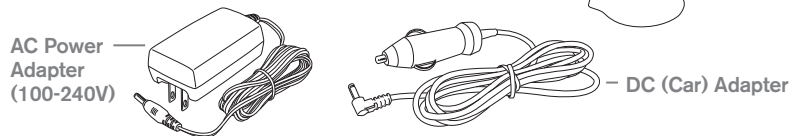
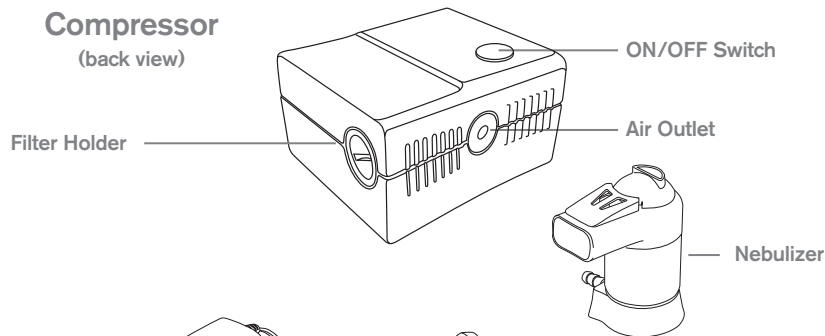
## MicroElite™

### MicroElite™ Compressor (back view)

Respironics MicroElite™ is a compressor nebulizer system powered by any of three power sources – AC power adapter, rechargeable battery, or DC car adapter for home medical use. The intended use of the MicroElite is to convert liquid medication into a fine mist that can be drawn into the lungs to relieve respiratory conditions for adult and pediatric patients who have been prescribed inhalation therapy or medication.

### Unpacking Your MicroElite System

When you unpack your MicroElite from its shipping case:

- Remove the compressor from the carrying case.
- Carefully examine the contents.
- Save these Instructions for Use for reference.
- Make sure you have all the content items listed and that they are not damaged.
- Report anything missing or damaged to Respironics.

Note: Retain carton and packing materials for use in shipping or storing the unit.

**Read these instructions before using this compressor.** If you do not understand any portion of these instructions, contact your healthcare provider or call Respironics Customer Service at 1-800-345-6443 or 1-724-387-4000.

**Read these WARNINGS AND CAUTIONS before using your MicroElite.**

### WARNINGS AND CAUTIONS

- ⚠ DANGER – To reduce the risk of electrocution:**
- Always unplug this compressor immediately after using.
  - Do not place or store product where it can fall or be pulled into a tub or sink.
  - Do not place in or drop into water or other liquid.
  - Do not reach for product that has fallen into water. Unplug immediately.
- ⚠ WARNING – To reduce the risk of burns, electrocution, fire or injury to persons:**
- Do not operate compressor while it is in carrying case.
  - A compressor should never be left unattended when plugged in.
  - Close supervision is necessary when this product is used by, on, or near children or physically challenged individuals.
  - Use this product only for its intended use as described in this manual. Do not use nebulizers or accessories not recommended by the manufacturer.
  - Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to a service center for examination.

- Do not use with an extension cord.
- Keep the power cord away from heated surfaces.
- Never block any opening of this product or place any objects into any product opening.
- This unit has been designed for intermittent use: 20 minutes ON/40 minutes OFF.
- To avoid the risk of possible electrical shock, the nebulizer must be operated using the specified power source.
- Do not remove the compressor housing or disassemble unit except to replace the filter. There are no serviceable parts.
- Never use while sleeping or drowsy.
- Do not use while operating a vehicle.
- Never drop or insert any object into any openings.
- Do not use where oxygen is being administered in a closed area such as an oxygen tent.
- Use the nebulizer that is supplied with the compressor. Failure to do so may compromise performance.
- Use only for purpose indicated in this instruction manual.

**SAVE THESE INSTRUCTIONS**

### **How To Use Your MicroElite**

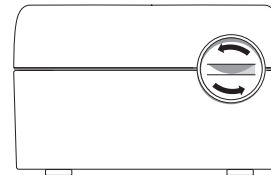
1. Check the medication nebulizer cup for damage. (The nebulizer enclosed with your MicroElite should be replaced regularly. Ask your healthcare provider for replacement guidelines and supplies.)
2. Place your MicroElite compressor on a firm, flat surface near an electrical outlet (if using AC power) and where you can be seated comfortably during your treatment.
3. To use the compressor with the AC power adapter, connect the AC power adapter into the power plug-in on the back of the compressor. Then connect the other end into the electrical outlet.
4. Alternatively the unit can be used with the (OPTIONAL) rechargeable battery pack or 12V DC (car) adapter. Read the instruction guide included with the rechargeable battery pack or DC (car) adapter carefully.
5. Do not operate the unit without the filter or with a clogged (dirty) filter. Operating the unit with a dirty filter can impair proper operation and cause damage that is not covered by the warranty.
6. Wash your hands before beginning your treatment.

7. Assemble your nebulizer and add your prescribed medicine following your prescription and the separate nebulizer instructions.
8. The nebulizer can be connected directly into the compressor or can be used with tubing by connecting one end of the tubing to the air outlet on the compressor.
9. Turn the power switch on  $\text{\textcircled{O}}$  to begin your treatment.
10. When your treatment is complete, turn the power switch off  $\text{\textcircled{O}}$  and unplug the unit if using AC power or DC (car) adapter.
11. Clean the nebulizer according to its instructions.

### **Care and Cleaning of Your MicroElite**

**CAUTION: To reduce risk of electric shock, DO NOT DISASSEMBLE and never submerge or place under running water.**

1. Unplug the compressor before cleaning.
2. Use a damp cloth to wipe down the outside of the compressor at least once each month (or more if necessary). Do not use abrasive cleaners.
3. The air filter must be replaced once a month or when dirty.
4. To change the filter:
  - Use a coin to unscrew the filter holder (Note: Turn COUNTER CLOCKWISE to unscrew).

**MicroElite Compressor  
(back view)**

- Pull out filter and replace with new filter.
  - Screw filter holder back on by turning CLOCKWISE.
5. After each cleaning, check that the unit is working properly by verifying that air still comes out of the nozzle when the unit is turned to the "on" position and that the unit's noise level has not increased. (Increased noise indicates improper filter positioning.)
  6. The compressor is oil-free. Do not lubricate.
  7. Store unit in a clean place when not in use.

## Troubleshooting

Problems	Solutions
Compressor overheats.	Turn the unit OFF for a minimum of 40 minutes to allow the compressor to cool.
The compressor does not operate.	Check all power sources to the compressor. Push the ON/OFF button again.  If you checked all the items above and the unit still does not work, allow the compressor to cool down for a minimum of 40 minutes before turning the compressor on again.
Medication does not nebulize properly.	Check for proper connection of the tubing adapter.  Make sure the nebulizer is not empty.  Clean the nebulizer as described in the manual.  Replace the nebulizer.  Check the intake filter and replace it if it is dirty.
The device is noisy.	Check that the filter cover is correctly positioned.

Should the device still not operate properly after checking the unit as indicated above, contact Respironics Customer Service at 1-800-345-6443 or 1-724-387-4000.

### Specifications:

Power Supply:	8.5VDC, 1.4A Max
Weight:	1.0 lbs. (with battery) 0.66 lbs. (without battery)
Size:	3" x 2-5/8" x 1-7/8" (without battery) 3" x 2-5/8" x 2-1/2" (with battery)
Maximum pressure:	17 PSI, 120 KPa
Maximum flow rate:	5 LPM
Mode of operation	20 minutes ON/40 minutes OFF
Class II Device (Double Safety Insulation).	
Type B Device (device with specific protection against electrical hazards).	
Certification:	
Electric Safety Standards EN60601-1	
Respiratory therapy equipment. Nebulizing systems and their components EN13544-1	
Electromagnetic Compatibility according to EN60601-1-2	
Storage Conditions:	
Temperature:	MIN -4°F (-20°C) – MAX 140°F (60°C)
Humidity:	MIN 10%RH – MAX 95%RH
Operating Conditions:	
Temperature:	MIN 41°F (5°C) – MAX 104°F (40°C) (with tubing) MIN 41°F (5°C) – MAX 77°F (25°C) (handheld)
Humidity:	MIN 10%RH – MAX 95%RH

### Replacement Parts and Accessories:

Filter	1041433
AC power/battery charger	1041517
DC Car Adapter (12V input/8.5V output)	1041515
Rechargeable Lithium Polymer Battery	RDD499
Pediatric Face Mask	1048154
Adult Face Mask	1048285
Micro Plus Nebulizer	1041431

### Symbol Glossary:

 Attention: Read accompanying documents	 ON/OFF (power)
 Alternating Current	 SN Serial Number

### Optional Rechargeable Battery:

Operating time with the battery	Up to 60 minutes
Number of treatments per charge	Up to 4 treatments when nebulizing 3 ml of medication
Normal time to charge	Approximately 3.0 hours
Charge cycle	Approximately 400 cycles

### Electromagnetic Information:

Portable and RF communication devices such as cellular phones, pagers, etc. can interrupt operation of electrical medical equipment. For this reason, your compressor must be placed far enough away from these devices to prevent interference. This device meets IEC60601-1-2 standard for electromagnetic compatibility (EMC).

EMC data sheets (1047862) are available upon request from Respironics, Inc. customer service at 1-800-345-6643 or 1-724-387-4000 or electronically at [www.microelite.respironics.com](http://www.microelite.respironics.com)

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## Warranty

Respironics, Inc. warrants the MicroElite compressor to be free from defects in materials and workmanship under normal use and operation for a period of 3 years from date of purchase from Respironics, Inc. The warranty is limited to repair or replacement, at Respironics, Inc.'s sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics, Inc. This warranty does not extend to any components or equipment subjected to misuse, improper operation, accidental damage, or unauthorized repairs, and is not extended to charges of, or for, labor repairs. All items returned must be properly packaged and shipped, prepaid, by the distributor servicing the unit. Respironics, Inc. shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

Made in China by Medel S.p.A. for Respironics New Jersey, Inc. MicroElite is a registered trademark of Respironics, Inc.  
©2007 Respironics, Inc. and its affiliates. All rights reserved.

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---

**RESPIRONICS®**

5 Wood Hollow Road  
Parsippany, NJ 07054  
www.microelite.respironics.com

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©2007 Respironics, Inc. et ses filiales. Tous droits réservés
©2007 de Respironics, Inc. y sus afiliadas. Todos los derechos reservados.