

## Mirage™ SoftGel

NASAL MASK

### Fitting Guide



1

- Turn the MicroFit dial to position 24, so that the forehead support is fully extended.



2

- Disconnect the lower headgear clip, hold the mask over the nose and pull the headgear over the head.



3

- Reconnect the lower headgear clip, ensuring it clicks into place.



4

- Adjust the upper straps evenly. Do not over-tighten.



5

- Adjust the lower straps to a comfortable position.



6

- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.



TIP

- Adjust dial to correct leak at nose.



TIP

- Adjust bottom straps to correct leak at top lip.

## Mirage™ SoftGel

NASAL MASK

### ConvertAble™ Comfort

The Mirage™ SoftGel Nasal Mask offers exceptional comfort with ResMed's renowned performance and quality. Easy and quick to fit, Mirage SoftGel is part of ResMed's exclusive ConvertAble™ Series, giving you the choice of two

mask cushions on one durable frame. Featuring a soft touch with excellent support, Mirage SoftGel adds the comfort, durability and freedom of choice that only a ConvertAble mask can offer.

**MicroFit dial**  
offers 24 positions for better fit, seal and comfort

**ResMed ConvertAble frame**  
accommodates both Mirage SoftGel and Mirage Activa™ LT cushions for business efficiency, cost savings and flexibility

**360° elbow rotation**  
positions the tubing for maximum comfort


**Quiet, diffused vent design**  
disperses air softly and quietly, away from bed partner

**Elbow partition**  
aids CO<sub>2</sub> washout and reduces noise output

**SoftGel's unique DoubleGel™ cushion**  
Clear gel layer provides ultimate comfort; blue gel layer provides support and stability

**Set-and-forget headgear clips**  
retain optimal headgear settings; easy to attach and detach

**Swivel**  
offers a convenient way to connect and disconnect air tubing

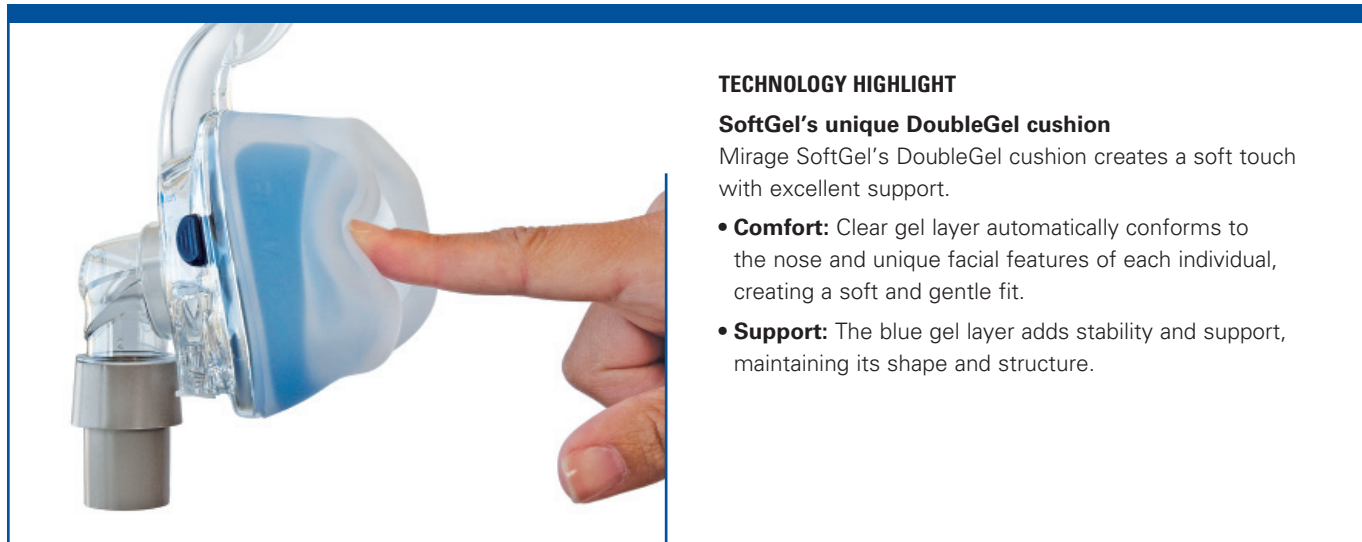

**TECHNOLOGY HIGHLIGHT**

**SoftGel's unique DoubleGel cushion**

Mirage SoftGel's DoubleGel cushion creates a soft touch with excellent support.

- **Comfort:** Clear gel layer automatically conforms to the nose and unique facial features of each individual, creating a soft and gentle fit.
- **Support:** The blue gel layer adds stability and support, maintaining its shape and structure.

**USER BENEFITS**

**Unparalleled softness and comfort**

- The DoubleGel cushion's outer layer provides unparalleled softness. The blue gel layer maintains stability and support
- Fitting is a simple task—for both current users and new patients

**Ultra-quiet therapy**

- Special array of small, built-in vents diffuse air quietly. That means a good night's sleep for patients and their bed partners

**Correct fitting made easier**

- Combination of the Mirage dual-wall cushion with the DoubleGel cushion design makes fitting simple
- The cushion adapts to individual facial contours, creating a comfortable fit with ease

**ConvertAble mask means freedom of choice**

- ResMed ConvertAble masks offer the choice of two different cushions on one frame
- Freedom of choice helps patients accept their therapy early and stay engaged in treatment

**PROFESSIONAL BENEFITS**

**ConvertAble means inventory efficiencies**

- Softgel is a multi-patient, multi-use mask suitable for repeated use in the lab
- Interchangeable and shared components promote inventory efficiencies and cost savings
- Fitting and training time is reduced
- Mirage SoftGel is extremely durable, ensuring consistent performance and fewer callbacks

**Customer satisfaction and added compliance**

- Increased therapy acceptance through first-time fit—long-term comfort
- Interchangeable cushions improve patient compliance while avoiding the cost of expensive mask replacements
- Choice of comfort and fit ensures patient satisfaction—minimizes callbacks
- Comprehensive user support package includes travel bag and user guide with mask fitting and care instructions

**PATIENT-PREFERRED**

- 8 out of 10 patients rate SoftGel better in comfort and durability than other gel masks<sup>1</sup>
- 3 out of 4 patients rated their sleep better with SoftGel<sup>1</sup>



**PRODUCT CODES**

**Mirage SoftGel Nasal Mask**

Medium	<b>61601</b>
Large	<b>61602</b>

**PRODUCT CODES**

**Reimbursement Codes**

Nasal mask use with positive airway pressure device, 1 per 3 months	<b>A7034</b>
Replacement cushion for nasal mask, 2 per month	<b>A7032</b>
Headgear used with positive airway pressure device, 1 per 6 months	<b>A7035</b>

**Mirage™ SoftGel**

NASAL MASK

**Components Card**

**Complete system**

M 61601                      L 61602

**Frame system\***

(no headgear)

**61627 (M)**  
**61628 (L)**

**Frame assembly\***


(no headgear, no cushion)

**60175 (St)**

**Elbow assembly**

(elbow, swivel)

**16399**

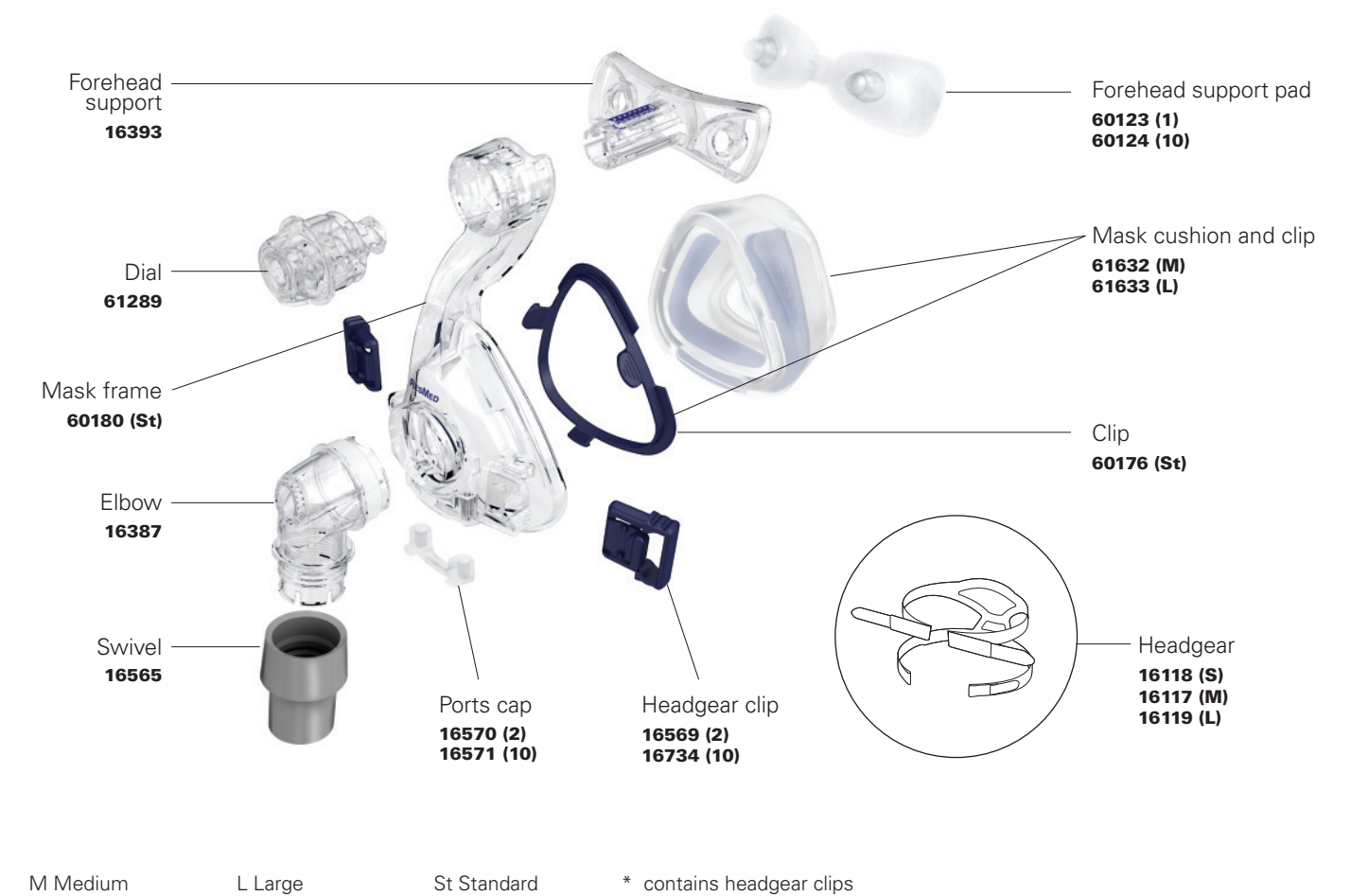

⚠ For details, refer to the *Mirage SoftGel Nasal Mask User Guide*.

<sup>1</sup> ResMed market research study of gel masks.